
## ergotron®

## LiFeKinnex™ CareFit™ Pro Firmware Update

for Devices Already Setup on eKinnex

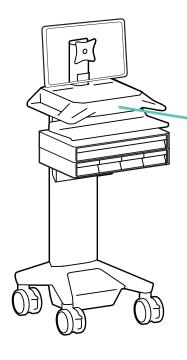

- 1 Please stand in front of the CareFit Pro cart and face the LCD display.
- **2** Power on the display (if not already powered on).



Click the settings button in the upper right corner.



4 Click "System Information".



5 If the CCU version has "Update Available" next to it, click the Update button.



The system performs the update and restarts. This typically takes less than 2 minutes.

For the latest User Installation Guide please visit: www.ergotron.com

For Warranty visit: www.ergotron.com/warranty

For Service visit: www.ergotron.com

For local customer care phone numbers visit: http://contact.ergotron.com

While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.