

ergotron®

LiFeKinnex™ Firmware Over-the-Air Update for 4-Bay Charger Connected to the Cloud


NOTE: These instructions are only valid for 4-Bay Chargers that have Inventek Wifi modules.

4-Bay Charger checks for firmware updates during power up or every 24 hours if plugged in. Firmware will update automatically when the following conditions are met:

1. All batteries need to be removed or fully charged when firmware update is available.
2. Charger needs to be plugged in and connected to WiFi.

Follow these instructions to manually run firmware update:

- 1** Remove all batteries from charger and unplug power cord before starting.
- 2** Plug in the power cord to turn ON charger.
- 3** Wait for device to connect to the WiFi.
Go to eKinnex web portal to verify the charger is connected to the WiFi.
eKinnex web portal: <https://lifekinnex.ekinnex.com/login>
If it doesn't connect to the WiFi, use the LifeKinnex setup tool to program network credentials on the device.
Set up tool: <https://www.ergotron.com/en-us/support/ekinnex/ekinnex-for-lifekinnex>
- 4** Firmware download will start automatically. Wait for download to complete (takes about 30 min).
The system will turn OFF, then back ON after the firmware update is complete.
- 5** Verify the updated firmware version on eKinnex for CareFit & LifeKinnex web portal.
<https://lifekinnex.ekinnex.com/login>

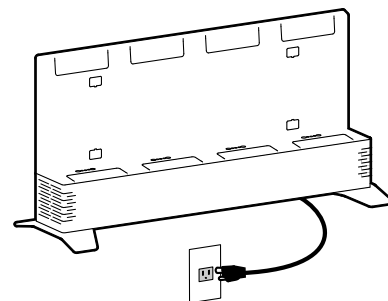

For the latest User Installation Guide please visit: www.ergotron.com

For Warranty visit: www.ergotron.com/warranty

For Service visit: www.ergotron.com

For local customer care phone numbers visit: <http://contact.ergotron.com>

www.ergotron.com | USA: 1-800-888-8458 | Europe: +31 (0)33-45 45 600 | China: 400-120-3051 | Japan: japansupport@ergotron.com

© 2023 Ergotron, Inc. All rights reserved. LiFeKinnex™ is a registered trademark of Ergotron, Inc.

While Ergotron, Inc. makes every effort to provide accurate and complete information on the installation and use of its products, it will not be held liable for any editorial errors or omissions (including those made in the process of translation from English to another language), or for incidental, special or consequential damages of any nature resulting from furnishing this instruction and performance of equipment in connection with this instruction. Ergotron, Inc. reserves the right to make changes in the product design and/or product documentation without notification to its users. For the most current product information, or to know if this document is available in languages other than those herein, please contact Ergotron. No part of this publication may be reproduced, stored in a retrieval system, or transmitted in any form or by any means, electronic, mechanical, photocopying, recording or otherwise without the prior written consent of Ergotron, Inc., 1181 Trapp Road, Eagan, Minnesota, 55121, USA Patents Pending and Patented U.S. & Foreign. Ergotron is a registered trademark of Ergotron, Inc.

888-06-119-G-00 rev. A • 04/23